WPT250

**ELECTRIC DOUBLE BREAST PUMP
ELEKTRİKLİ ÇİFTLİ GÖĞÜS POMPASI**

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USER MANUAL

EN

KULLANIM KILAVUZU

TR

www.weewell.com
info@weewell.com

CONGRATULATIONS

Dear Stakeholder,

We congratulate you on having one of the Weewell products which are safely preferred all over the World and becoming a part of the Weewell family. We are pleased to serve you with our carefully designed products that appeal to babies such as video monitor, audio monitor, thermometer, nursing and feeding group products.

Please read the user manual carefully before using the product. This guide has been meticulously prepared to get the most out of your purchased product. You can call our call center to get all kinds of information about the device and to contribute to our family with your suggestions and opinions. If you face with any problems or malfunctions, you can contact Weewell authorized service. For safety of yourself and other persons, read the **SAFETY INSTRUCTIONS** before using the device.

Best Regards,
Weewell Turkey Team

CONTENTS

PRODUCT OVERVIEW.....	3
SAFETYINSTRUCTIONS.....	4
PARTS AND ASSEMBLE	6
OPERATIONSINSTRUCTIONS.....	8
CLEAINING AND DISENFECTION	10
SHIPPINGAND DISPOSAL.....	11


PRODUCT OVERVIEW

Weewell Electric Double Breast Pump provides a natural and maximum comfort milking experience with its Touch & Sense technology. The soft massage pad design imitates the sucking movement of the baby with the soft and warm feeling it provides. It is allowed to flow quietly, comfortably, gently and quickly. The breast pump is compact, easy to assemble, and its components are bisphenol A-free and dishwasher safe.

As lactation experts say, breast milk is the best food for babies under one year old. Breast milk is the most suitable food for the needs of the baby with its content and the antibodies in the milk protect the baby against infections.

The breast pump helps you to breastfeed your baby for longer. Even if you cannot breastfeed your baby yourself, your baby has the opportunity to enjoy breast milk at all times. With its mobile design, easy portability and effective milking performance, you can carry your breast pump anywhere, and easily perform the milking process wherever you want.

Schematic Diagram of Motor

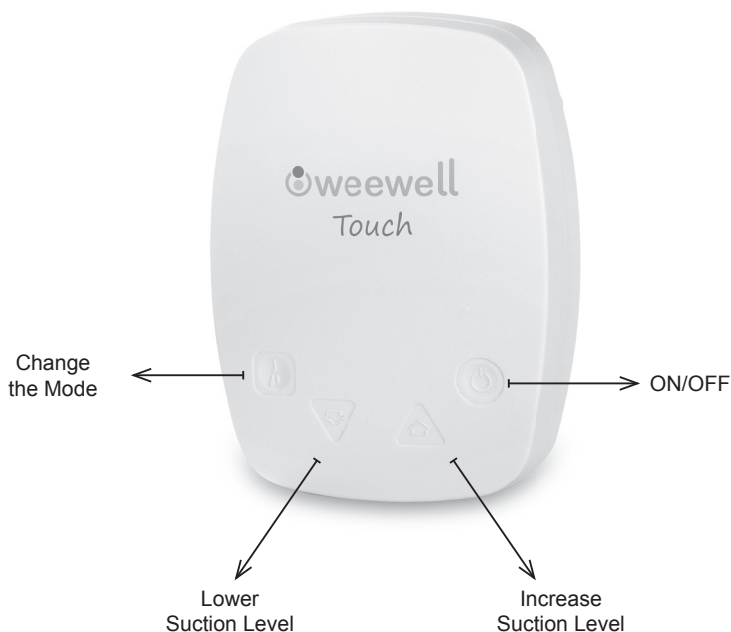Notice: Use the power supply of 5V 1A

SAFETY INSTRUCTIONS

(Information about usage errors, maintenance & repair)

- 1- Weewell breast pump parts should be used after they have been assembled completely and without errors as described in the user manual. Otherwise, the product will not work properly and the desired efficiency will not be provided.**
- 2- Read the "Operation Instructions" and "Usage Methods" in the user manual carefully and apply them completely. Otherwise, the milking process will not be performed properly.**
- 3- By following the instructions on "Storage of Milk" in the user manual, you can make sure that the milk you express is healthy for your baby.**
- 4- This is a product with a high degree of hygiene. Therefore, this kit is totally personal; sharing it with others is not suitable for health.**
- 5- Use the product with its original adapter. In case of deterioration or loss, supply it from the technical service.**
- 6- Do not demount the product or do not try to repair it. If your device breaks down, take it to Weewell Technical Service for repair. Disassembling the device, performing operations on it, or interfering with the system outside the authorized service will cause the device to be out of warranty.**
- 7- During use or storage, keep the product away from moisture, dust, direct sunlight, any heat source and devices that emit heat. Products that deteriorate by exposure to heat, moisture or dust are not covered by the warranty.**

Warnings

- Please consult a medical specialist about the suitability of your breast structure to use a pump.**
- Never immerse the motor and adapter in water.**
- Only use original parts, otherwise your device may be damaged.**
- If the device breaks down, it should only be repaired by the Authorized Service, otherwise it will be out of warranty.**
- If it is used for a premature baby, be sure to follow the doctor's recommendations.**
- Before using, make sure that all parts are clean and sterile.**


General Descriptions

It is recommended that you are located in a quiet, comfortable place where no one will distract you while milking. It is quite normal that it takes a while for the milk to come in when using it for the first time. Your body will adjust milk production according to daily consumption. Be aware that daily milk consumption varies from baby to baby. For this reason, it is recommended to try to increase fluid consumption rather than trying to increase milk production via several methods during the first breastfeeding period.


Breast Massage – Preparation for Milking

Some mothers may normally have difficulty in milking with a manual or electric breast pump. Applying a warm compress on the breast will open the milk ducts and ease the flow of milk. Prepare the hot compress as follows:

- 1- Soak a clean towel in hot water. (60-70°C)
- 2- Remove excess water by squeezing the towel.
- 3- Place the towel on your chest and hold it for a few minutes until it cools down.
- 4- Repeat all the above steps four times.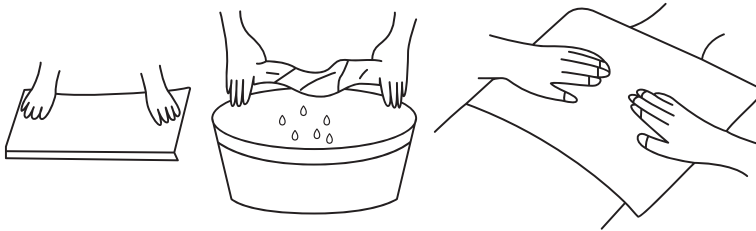

- Applying a chest massage after the hot compress is completed, it will help open the chest ducts.

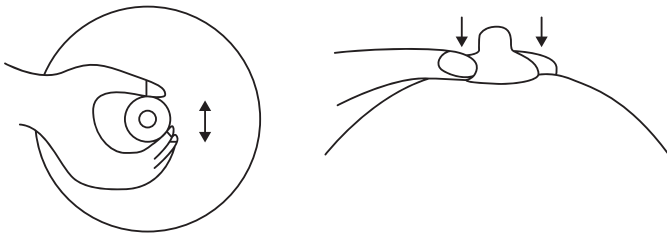- Put your thumb and forefinger on the tip of the chest and gently massage 5-6 times up and down.


- Gently massage the chest with circular movements by placing your four fingers and thumb as in the figure. Stop the move after 5-6 turns for each chest.

Important note: If you feel pain or uncomfortable during the hot compress or massage, stop it immediately and consult a doctor.

PARTS AND ASSEMBLE

- | | | | |
|-------------------|--------------|-----------|------------------|
| 1- Feeding Bottle | 2- Valve | 3- Nipple | 4- Breast Shield |
| 5- Diaphragm | 6- Connector | 7- Tube | 8- USB Cable |

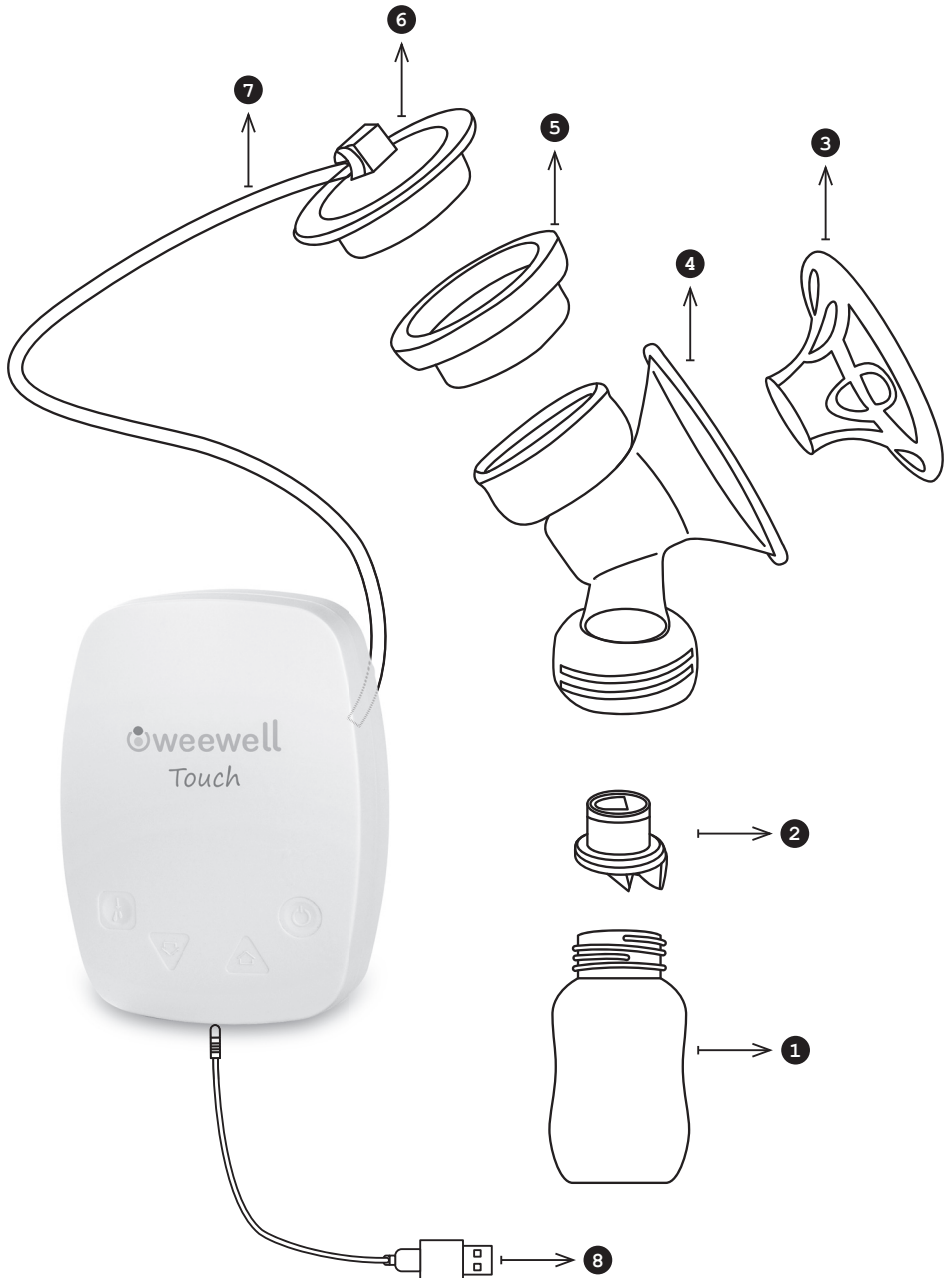How to Assemble

Before the first use, all components of the breast pump should be cleaned and disinfected in accordance with the chapter "cleaning and disinfection". All components need to be cleaned after each use, and all components must be disinfect before each use.


Pump Assembling

Notice: make sure that you Make sure you have all the components of the breast pump cleaned and disinfected before touching the cleaned parts, please wash hands thoroughly. Be careful that the cleaned parts may still be hot.


Before assembly, please disinfect pump parts, wash hands thoroughly

Tips: You may find it easier to assemble your breast pump when it is wet.

Please Read the Installation Diagram Carefully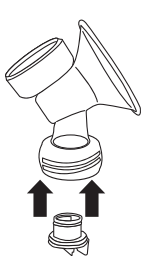

1- Insert the valve into the bottom of the breast shield.

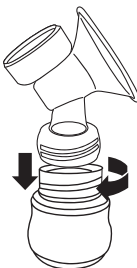2- Screw the breast shield onto the feeding bottle until it is fully fixed.

3- Put the diaphragm into the top part of the breast shield. Press the diaphragm down to ensure it's stable.

4- Attach the connector to the breast shield. Connect one of the tube to the connector and the other side to the motor.

5- Connect the USB to the motor to power the device.

OPERATION INSTRUCTIONS

- 1- Connect the power supply and touch the power button to start the pump.
- 2- Touch the mode button to choose massage or pump mode.
- 3- According to your need, touch + or – button in the appropriate mode to adjust the frequency and suction of pumping.
- 4- The default working time is 30 minutes, beyond which the breast pump will automatically shuts down or touch the power button to turn off the power.
- 5- Breast milk should not be filled over the valve.

Usage Method

- 1- Wash your hands nicely and keep your chest clean before pumping. Relax and make sure you are sitting in a comfortable chair (you can use a cushion to support your back). It is recommended to have a glass of water with you.
- 2- Hold your chest gently and place it on the shield. Make sure your nipples are centered.
- 3- When the red light is on, tap + - to select the massage intensity that is more proper for you and when the breast milk starts to flow, switch to the expressing mode. The blue light will turn on. Adjust the most suitable suction power and frequency for you by the mode button.

Warning: Maximum suction mode does not guarantee the best performance. Use it in the mode you feel most comfortable.

- 4- When milking is complete, turn off the breast pump and then carefully remove the breast pump from the breast.
- 5- Remove the bottle from the breast pump to switch to maintenance or storage condition. Clean used breast pump components by following the instructions in the "cleaning and disinfection" section.
- 6- Unplug the motor.

Attention and Warning

This product is a personal product, please don't share, so as not to affect health. When removing or connecting parts, operate carefully, don't overexert, so as not to cause damage. This product should not be exposed in the hot sun, and should be placed out of children's reach.

Single / Double Suction Operation


Single Suction

Connect one of the tube to the connector and the other side to the motor. After connecting, you can start your operation via the motor unit

Double Suction

First, assemble the tube of motor connection end with the double connecting apparatus. Then connect the ends of the apparatus to the chest shields. Once the connection is established, you **can start your operation via the motor unit**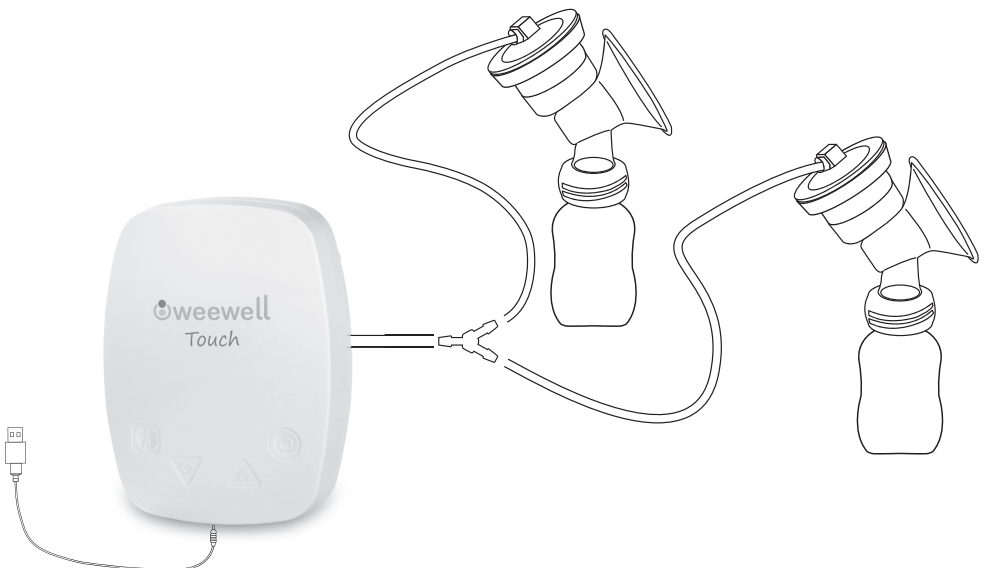

CLEANING AND DISINFECTION

Before first use, please clean and disinfect all parts of pump. Wash all the parts every time before your use. Completely remove the breast pump. The breast pump should be removed completely, as well as the duckbill valve.

Be careful when removing and cleaning the duckbill valve. If the duckbill valve is damaged, the breast pump will not work properly. To remove the duckbill valve, gently pull the small piece on the side of the duckbill valve. Clean all the components with hot water and a little detergent, and rinse thoroughly.

To clean the duckbill valve, please rub it gently in the warm water with a few drops of cleaning liquid.

Don't insert any object into the duckbill valve, which may cause damage. Disinfect all the components by using a special sterilizer or boiling in boiling water for 5 minutes.

Warnings for cleaning

- After using, please wipe the motor via a damp cloth.
- Do not clean the motor, adapter and adapter cable via a damp cloth, do not immerse in water.
- You can sterilize parts other than motor, adapter and USB cable by cleaning them.
(Important Warning: It is recommended to disinfect plastic and silicone parts by using a sterilizer. Boiling method will probably damage plastic and silicone parts and render them unusable.)
- Ensure that all parts are installed correctly when assembling or disassembling. Be careful not to lose small parts. Missing parts may cause the product to malfunction.
- Before cleaning, unplug the cable and wait until the device cools down.

Cleaning instruction

- 1- Unplug the adapter from the socket, disconnect the cable and then wait for the device to cool down.
- 2- Demount all parts. Wipe the adapter, motor, cable only via a slightly damp cloth.
- 3- Clean and rinse the other parts thoroughly with soapy water.
- 4- Sterilize the parts (except the motor, adapter and cable) in the sterilizer. Boiling is not recommended because of the possibility of damage to the parts and the lack of hygiene.
- 5- Remount after the parts are completely dry. If you do not use it for a while, keep it in its box.

Attention!

In cases where plastic parts are damaged due to misuse, they are out of warranty.

Using the product despite the loss of parts causes the product to be out of warranty.

Lost parts cannot be considered under warranty.

When applying for repair, an invoice and a signed, stamped warranty document must be declared to benefit from the warranty.

Out of warranty situations

Misuse, parts be exposed to impact and damage

Warranty card lost or damaged

All kinds of damage and deterioration caused by the user

STORAGE AND HEATING OF BREAST MILK

Breast Milk Storage

After the breast milk is absorbed, close the storage cover tightly or seal the storage bag. The milk must be put into the refrigerator immediately. Breast milk can arbitrarily be refrigerated or frozen by the refrigerator. It can be stored by refrigerated for 48 hours or frozen for 3 months.

Breast Milk Heating

After the milk melts, warm it via a milk warming device or warm water. Do not use a microwave or boiling water to heat milk or boil the milk, so as to avoid the loss of nutrient. After heating, your baby can be fed.

SHIPPING AND DISPOSAL

Handling and Shipping

The handling and transportation of the products is important to affect the usage life of the products. It is needed to be extremely careful when transporting and transporting the products you buy. Damages that may occur as a result of falling and impact will definitely be considered out of warranty. In addition, when you want to take the product out of the house or office you are using, definitely place it carefully in its original box and carry it.

The original packaging will relatively protect the product against impacts and falls. When you need to send the product to our authorized services for service, send it in its original box and, if possible, wrap the original box with a shock-absorbing bubble bag or similar materials to strengthen the package so that it will not be damaged during shipping.

Energy

The product works with USB energy sources with 5V/1ADC output.

Technical Specifications

Model:	Weewell
Prodcut name:	Electric Double Breast Pump
Lot No: 2-20223007	Model: WPT250
Bench life:	5 Years
Operating Temperature:	>5

Disposure

When disposing of this product and its parts, please take them to an appropriate recycling point, taking into account the environmental effects. If possible, the materials must be separated into plastic and metal during recycling.

For more details, contact technical service.

Do not forget! Protecting our environment is our duty.

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CIXI RUIHONG ELECTRIC APPLIANCE CO.,LTD

ADDRESS: 18 Xiamaojia Rd, Xinpu, Cixi City, Ningbo, Zhejiang, China
TEL NO. +8657463581765 FAX NO. +8657463581761

EC Declaration of Conformity

We, CIXI RUIHONG ELECTRIC APPLIANCE CO.,LTD

Declare under our sole responsibility that the product (s):

Description : Double Electric Breast Pump

Brand name : WEEWELL

Model Number : WPT250

To which this declaration relates is in conformity with the essential requirements and other relevant requirements of EMC Directive 2004/30/EU. The product is in conformity with the following standards and/or other normative documents.

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