WTA300

INFRARED THERMOMETER
KIZILÖTESİ TERMOMETRE

ATEŞ ÖLÇEN CİHAZLAR

USER MANUAL

EN

KULLANIM KILAVUZU

TR

www.weewell.com
info@weewell.com

CONGRATULATIONS

Dear Stakeholder,

We congratulate you on having one of the Weewell products which are safely preferred all over the World and becoming a part of the Weewell family. We are pleased to serve you with our carefully designed products that appeal to babies such as video monitor, audio monitor, thermometer, nursing and feeding group products.

Please read the user manual carefully before using the product. This guide has been meticulously prepared to get the most out of your purchased product. You can call our call center to get all kinds of information about the device and to contribute to our family with your suggestions and opinions. If you face with any problems or malfunctions, you can contact Weewell authorized service. For safety of yourself and other persons, read the SAFETY INSTRUCTIONS before using the device.

Best Regards,
Weewell Turkey Team.

CONTENTS

1. INTENDED USE / INDICATIONS FOR USE.....	2
2. BEFORE START.....	2
3. PRODUCT INFORMATION.....	2
4. CLASSIFICATION.....	2
5. SAFETY PRECAUTIONS.....	3
EXPLANATION OF MARKS OR SYMBOLS.....	3
WARNING.....	4
6. DISPOSAL	5
7. IMPORTANT SAFETY INSTRUCTIONS.....	5
8. HOW THIS THERMOMETER MEASURES TEMPERATURE	6
9. UNIT DESCRIPTION.....	7
10. DISPLAY.....	7
11. DIRECTIONS FOR USE.....	8-10
11.1 Ear Temperature Mode.....	8
11.2 Forehead Temperature Mode.....	9
11.3 Object Temperature Mode.....	10
12. NOTES.....	10
13. CHANGING BETWEEN CELSIUS AND FAHRENHEIT.....	11
14. HOW TO RECALL READINGS IN MEMORY MODE.....	11
15. ERROR MESSAGES.....	12
16. REPLACE BATTERIES.....	12
17. MAINTENANCE, STORAGE AND CALIBRATION.....	13
18. TECHNICAL SPECIFICATIONS.....	14
19. INCLUDED IN DELIVERY.....	15
20. EMC INFORMATION.....	15-17

1. INTENDED USE / INDICATIONS FOR USE

The Infrared Thermometer is intended for measuring human body temperature of people aged 3 months and above by detecting infrared heat from ear canal or non-contact (about 1.5-5.0cm distances) with the forehead in the home or hospital environment. Contraindications: no.

2. BEFORE START

- For safe and proper use of this product, be sure to read and fully understand Safety Precautions contained in this user manual.
- If it is not convenient to measure the forehead or ear canal, please refer to the measurement at the back of ear.
- Keep this manual with you for a convenient reference.
- Keep this manual in a proper way and avoid any loss.
- If you need other information, please contact the manufacturer (See the Product Information).

3. PRODUCT INFORMATION

- Product name: Infrared Thermometer,
- Product No: WTA300
- Model: AET-R612A
- Software version: A.01
- Standard: The product is made under the ISO80601-2-56.

4. CLASSIFICATION

1. Internally powered equipment;
2. Type BF applied part;
3. IP classification: IP22;
4. Disinfection of 70%-80% medical alcohol;
5. Not Category AP / APG equipment;
6. Mode of operation: continuous operation.

5. SAFETY PRECAUTIONS

- Warning marks and symbols are contained for your safe and proper use of this product and prevention of any injury to you and others.
- See Table below for description of warning marks and symbols:

 WARNING	Means a possibility of personal injury in case of improper use.
 NOTICE	Means a possibility of personal injury or property damage in case of improper use.

[* Property damage covers any damage to house, family property, domestic animal and pet.]

EXPLANATION OF MARKS OR SYMBOLS

The following symbols may appear on the manual, Infrared Thermometer and its accessories

	MUST OBSERVE Means Obligatory with detailed items expressed in words or figures within or beside the mark. Left one means General Compulsory.
	Follow instructions for use.
	IMPLICATION OF SYMBOL Type-BF applied part
	Caution: Consult accompanying documents.
	Transport package shall be kept away from rain.
	Transport package shall not be exposed to sunlight.
	Indicates correct upright position of the transport package.
	Contents of the transport package are fragile therefore it shall be handled with care
	Indicates temperature limits within which the transport package shall be stored and handled.
	Non-ionizing electromagnetic radiation
	Do not roll
	Lot number
	Production date
	The device should not be used after the end of the shown or the day
IP22	Protected against solid foreign objects of 12.5 mm diameter and greater. protected against vertically falling water drops when the device is tilted up to 15°.
	Manufacturer
CE 0123	CE Mark: indicates that the device complies with the EU 2017/745
	Authorized Representative in the European Community
	Medical Device

WARNING

- Do not use the thermometer under temperature extremes (below 15°C/59°F or over 40°C/104°F) or humidity extremes (below 10%RH or over 80%RH). * Failing to do so may cause inaccuracy.
- Do not expose the thermometer to temperature extremes (below -20°C /-4°F or over 55°C/131°F) or humidity extremes (below 15%RH or over 93%RH). * Failing to do so may cause inaccuracy.
- Temperature probe has been shocked and lax. *Contact your retailer immediately.
- It is dangerous for patients to perform a self-evaluation and self-treatment based on the measuring results. Be sure to follow doctors' instruction. * A self-evaluation may cause deterioration of diseases.
- Don't touch or blow infrared sensor.
 - * A polluted or broken infrared sensor may cause inaccuracy.
- Clean a polluted infrared sensor with a soft dry cloth in a gentle manner.
 - * Clean with toilet tissue or paper towel may scratch the infrared sensor, causing inaccuracy.
- Install the battery in the right position according to the manual.
 - * Incorrect replacement will cause battery heat.
- Do not immerse the thermometer in liquids.
 - *This instrument is not waterproof.
- The user must check that the equipment functions safely and see that it is in proper working condition before being used.
- Do not use a mobile phone nearby when the thermometer is working.
 - *Portable and mobile RF communications equipment can affect MEDICAL ELECTRICAL EQUIPMENT. Failing to do so may cause inaccuracy.
- The probe glass is made from special material which is not easy broken.
 - *If there is any damage, stop using it and contact the retailer immediately.
- Do not modify this equipment without authorization of the manufacturer.
 - *Contact the retailer immediately.
- The device is not repairable and contains no user serviceable parts.
 - *If there is any problem, contact the retailer immediately.
- The device requires no calibration.
 - *If there is any problem, contact the retailer immediately.
- No modification of this equipment is allowed.
 - *If there is any problem, contact the retailer immediately
- If this equipment is modified, appropriate inspection and testing must be conducted to ensure continued safe use of the equipment.
 - *If there is any problem, contact the retailer immediately.
- The operator shall not touch the thermometer and the patient simultaneously.
- Keep the machine out of children's reach.
 - * For accidental swallow of battery or protective film, please consult the doctor at once.
- For temperature difference between storage area and measurement site, condition the thermometer for about 30minutes in room temperature (measurement site).
 - * Failing to do so may cause inaccuracy.

WARNING

Don't throw batteries into fire.
* Battery is likely to explode in fire.

RECOMMENDATIONS

- Don't use the product for other purposes.
- The patient can use and maintenance the device as an operator.
- Children less than 11 years old use this unit must be accompanied by adult.
- It is not waterproof device. Clean it only with damp cloth gently.
- It is forbidden to leave the product exposed to any chemical solvent, direct sunshine or high temperature.
- Don't drop, tramp or impose any vibration or impact on the product.
- Take out the batteries if you are not going to use the unit for a long period of time.

6. DISPOSAL

Do not dispose of electrical appliances as unsorted municipal waste, use separate collection facilities. Contact your local government for information regarding the collection systems available. If electrical appliances are disposed of in landfills or dumps, hazardous substances can leak into the groundwater and get into the food chain, damaging your health and well-being.

7. IMPORTANT SAFETY INSTRUCTIONS

- This instrument may only be used for the purposes described in this booklet. The manufacturer cannot be held liable for damage caused by incorrect application.
- Never immerse this instrument in water or other liquids. For cleaning please follow the instructions in the Cleaning and Disinfecting section.
- Do not use the instrument if you think it is damaged or notice anything unusual.
- Never open the instrument.
- The device is not suitable for use in the presence of flammable anesthetic mixtures with air or with oxygen or nitrous oxide.
- A basic physiological effect called vasoconstriction can occur in the early stages of fever, resulting in a cool skin effect. The recorded temperature using this thermometer can, therefore, be unusually low.
- If the measurement result is not consistent with the patient's finding or unusually low, repeat the measurement every 15 minutes or double check the result by another core body temperature measurement.
- This instrument comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the Technical Specifications section.
- Ensure that children do not use the instrument unsupervised; some parts are small enough to be swallowed.

- Protect it from:
 - extreme temperatures
 - impact and dropping
 - contamination and dust
 - direct sunlight
 - heat and cold
- If the instrument is not going to be used for a prolonged period the batteries should be removed.

WARNING


- **Use of this instrument is not intended as a substitute for consultation with your physician.**
- **This instrument is NOT waterproof so do not immerse in liquids.**

8. HOW THIS THERMOMETER MEASURES TEMPERATURE


This thermometer measures infrared energy radiated from the ear canal or forehead . This energy is collected through the lens and converted to a temperature value.

Temperature readings obtained by scanning the area will provide the greatest accuracy.
provide the greatest accuracy.

9. UNIT DESCRIPTION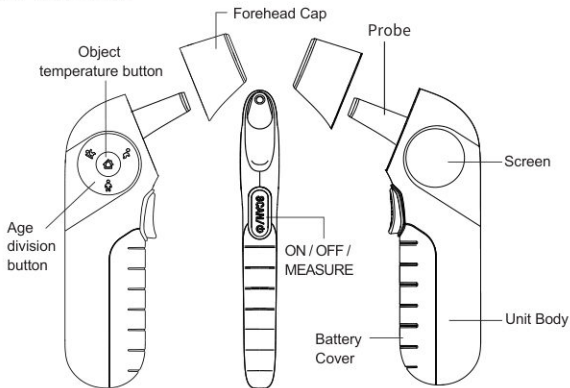

10. DISPLAY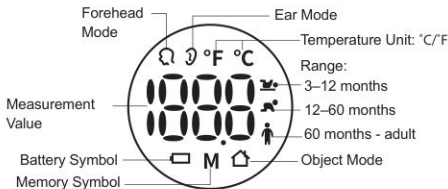

Forehead mode: The icon indicates currently in Forehead temperature measurement mode.

Ear mode: The icon indicates currently in Ear temperature measurement mode.

Temperature unit: Unit of measuring temperature shows °C or °F.

Measurement value: The temperature reading is displayed after measuring.

Range: Indicates division of different age groups.

Battery symbol: The icon prompts to replace the battery when the power is low.

Memory symbol: It shows in memory query mode currently.

Object mode: Indicates currently in object temperature measurement mode.

11. DIRECTIONS FOR USE



Press "SCAN / $\cup$ " button to turn on the thermometer, the screen will be activated, then the most recent measurement value will be displayed. When the beep sound is heard and the screen shows —C, indicating that the thermometer is ready for temperature measurement. There are three measurement modes: Ear/Forehead/Object.

Select a measurement mode

11.1 Ear Temperature Mode

Remove the forehead temperature cap, the "D" symbol is displayed. Use the age division button to select the appropriate age group, then fit the probe properly into the ear canal and press the "SCAN / $\cup$ " button, the thermometer will beep and display the measuring temperature.

When taking continuous repeated measurements, remove the probe between each measurement and pause for at least 5 seconds before proceeding to the next one. This will ensure more accurate readings.

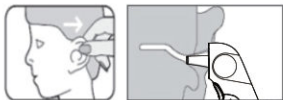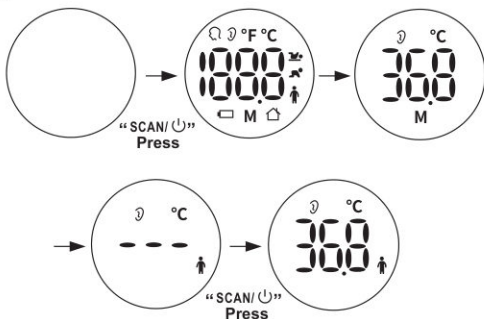

For children under 12 months, please pull the outer ear back gently:

For children over 12 months and adults, please pull the outer ear back and upward

11.2 Forehead Temperature Mode

Cover the forehead temperature cap, the "Ω" symbol is displayed. Use the age division button to select the appropriate age group, then take the thermometer to aim towards the forehead about 1.5-5cm distances and press "SCAN / ⏻" button, the thermometer will beep and display the measuring temperature. The next measurement can be taken after about 5 seconds.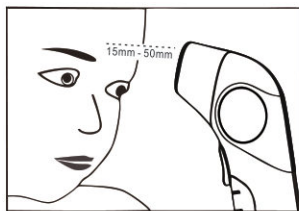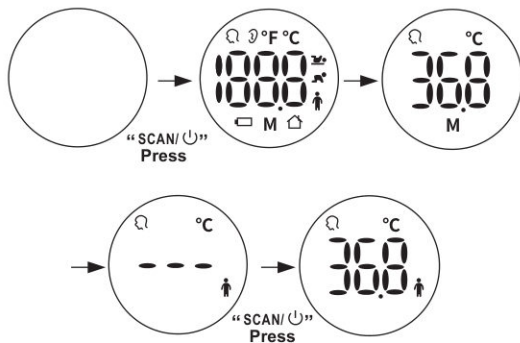


The correct measurement method to follow:

Measure the forehead.

Distance 1,5 cm to 5 cm

11.3 Object Temperature Mode

Press the object temperature button " " to select the object mode, when " " is displayed, aim the thermometer at the object and press "SCAN /  " button, the thermometer will beep and display the measuring temperature. The next measurement can be taken after about 5 seconds.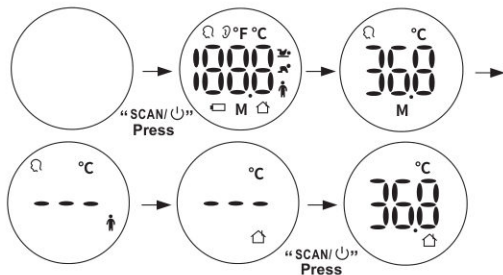


Press and hold the "SCAN /  " button for about 5 seconds to turn off the thermometer. Or the device will switch off automatically after 60 seconds of inactivity.

12. NOTES

- Patients and thermometer should stay in steady state room condition for at least 30 minutes.
- Don't take a measurement while or immediately after nursing a baby.
- Patients should not drink, eat, or exercise before/while taking the measurement.
- Don't move the measurement device from the measuring area before hearing the termination beep.
- Use an alcohol swab to carefully clean the probe and wait for 15 minutes before taking a measurement on another patient.
- Always take the temperature in the same location, since temperature readings may vary according to location.
- In the following situations it is recommended that three temperatures are taken and the highest one taken as the reading:
 1. Children under three years of age with a compromised immune system and for whom the presence or absence of fever is critical.
 2. When the user is learning how to use the thermometer for the first time until he/she has familiarized himself / herself with the instrument and obtains consistent readings.
 3. If the measurement is surprisingly low.
- Readings from different measuring sites should not be compared as the normal body temperature varies by measuring site and time of day, being highest in the evening and lowest about one hour before waking up.


13. CHANGING BETWEEN CELSIUS AND FAHRENHEIT

In shutdown mode, press and hold the "SCAN / $\cup$ " button for about 8 seconds until "—" is displayed, then the current temperature unit will show, press the "SCAN / $\cup$ " button to select $^{\circ}\text{C}$ or $^{\circ}\text{F}$. The thermometer will automatically exit by not pressing any button in about 4 seconds.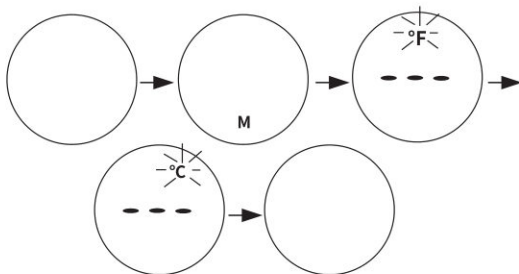

14. HOW TO RECALL READINGS IN MEMORY MODE

In shutdown mode, press and hold the "SCAN / $\cup$ " button for about 4 seconds, the thermometer enters Memory mode, the most recent reading will be displayed. Press the "SCAN / $\cup$ " button again to display the next reading from the last to the first in sequence. The thermometer stores the last 32 sets temperature measurements.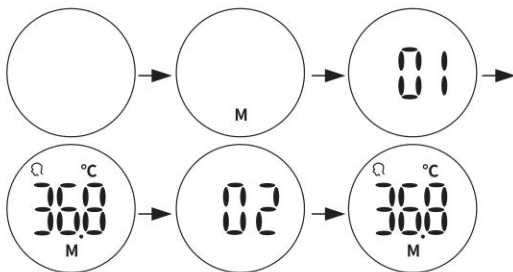


15. ERROR MESSAGES

Error message	Problem	Solution
Hi	Temperature is higher than 42.2°C(108°F).	Operate the thermometer only between the specified temperature ranges. In the event of a repeated error message, contact your retailer or Customer Services
Lo	Temperature is lower than 32°C(89.6°F).	Operate the thermometer only between the specified temperature ranges. In the event of a repeated error message, contact your retailer or Customer Services
Err	Sensor not welded	Contact your retailer or Customer Services
ErE	EEPROM is abnormal	Contact your retailer or Customer Services
	Low power	Replace two new batteries

16. REPLACE BATTERIES

When the low battery symbol " " is indicated, please replace new battery. The steps are as follows:

1. Hold the thermometer with your hand as follows.
2. Press firmly on the battery cover with your fingers, then push in the direction of the arrow until the battery cover opens, as shown in the following figure.
3. Remove the used batteries and install two new AAA batteries, making sure the poles are in the right direction.
4. Close the battery compartment.

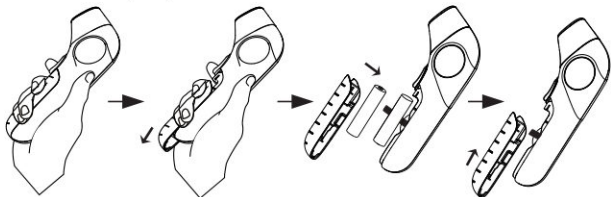WARNING

- Please take out the battery if you do not use the thermometer for a long time.
- Please refer to the relevant laws and regulations for the disposal of waste batteries.
- Do not mix old and new batteries;
- Do not mix alkaline, standard (carbon-zinc), or rechargeable (Ni-Cd, Ni-MH, etc) batteries.

17. MAINTENANCE, STORAGE AND CALIBRATION

Maintenance:

- Remove any stains on the body with a soft and dry cloth.
- Gently wipe the dirt on the body with a soft cloth, use a cotton swab or soft cloth stained with alcohol, and gently wipe the lens of the temperature measuring head. After using the product, the product can be sterilized with 70%- 80% medical alcohol on the soft towel or cotton sliver.
- If the probe tip is damaged, please contact the maintenance center.
- Don't wash with water or detergent containing abrasive or benzene.
- Do not immerse in liquids.

Storage:

- The device must not be stored or used at an excessively high or low temperature or humidity, in sunlight, in association with an electrical current or in dusty locations. Otherwise may occur inaccuracies.
- Remove the battery if non-use for long time.

Calibration:

- This thermometer is calibrated at the time of manufactured. If the thermometer is used according to the instruction, periodic re-calibration is not required. If at any time you question the accuracy of the measurement, please contact the retailer immediately.
- Don't attempt to modify or reassemble the thermometer.

18. TECHNICAL SPECIFICATIONS

Device Name	Infrared Thermometer
Measurement Mode	Forehead mode, ear mode and object temperature mode. The ear and forehead temperature measurement mode the object temperature mode is direct mode
Measurement Site	Forehead and ear canal
Reference Body Site	Oral
Measurement Unit	Celsius (°C) and Fahrenheit (°F)
Measurement Range Forehead and Ear Mode	32,0°C – 42,2°C (89,6°F – 108°F)
Accuracy (Laboratory)	±0.2°C (35.0°C–42.0°C) / ±0.4°F (95°F–107.6°F), ±0.3°C (±0.5°F) other range
Clinical Repeatability	≤ ±0,3°C (0,5°F)
Accuracy for Clinical Test	≤ ±0,3°C (0,5°F)
Resolution of Display	0,1°C / 0,1°F
Measuring time	1 s (5 s for another measurement)
Memories Recall	32 measurements recall
Battery	2 x AAA batteries(DC 3V)
Battery Life	New batteries (2 x AAA batteries) will take approx 1000 measurements
Automatic shutdown function	60s ±10s
Product Life	5 Years
Operating Conditions	15°C–40°C (59°F – 104°F) 10% RH–80% RH, 70kPa–105 kPa
Storage and transportation condition	-20°C–55°C (-4°F – 131°F) 15% RH–93% RH, 70kPa–105 kPa
Time of thermometer from highest to lowest temperature environment to operation environment	30 Minutes
High Temperature Hint	≥ 37.8°C (100°F)
Dimensions	154 mm x 67 mm x 19 mm
Weight	57 g Without batteries
Software version	A.01

Note: Not intended to be sterilized
 Not for use OXYGEN RICH ENVIRONMENT
 Not intended for use in the emergency medical services environment.
 Not intended for use with flammable agents.
 Not intended for use with flammable anesthetics

19. INCLUDED IN DELIVERY

Infrared thermometer X1 Instruction for use X1

Version:1.0 Date modified: 2025-10-17

Please read this instruction manual before use

20. EMC INFORMATION

The ME EQUIPMENT or ME SYSTEM is suitable for home or hospital environment.

Warning: Don't near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances is high.

Warning: Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Warning: Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation."

Warning: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of Infrared Thermometer, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

If any: a list of all cables and maximum lengths of cables (if applicable), transducers and other ACCESSORIES that are replaceable by the RESPONSIBLE ORGANIZATION and that are likely to affect compliance of the ME EQUIPMENT or ME SYSTEM with the requirements of Clause 7 (EMISSIONS) and Clause 8 (IMMUNITY). ACCESSORIES may be specified either generically (e.g. shielded cable, load impedance) or specifically (e.g. by MANUFACTURER and EQUIPMENT OR TYPE REFERENCE).

If any: the performance of the ME EQUIPMENT or ME SYSTEM that was determined to be ESSENTIAL PERFORMANCE and a description of what the OPERATOR can expect if the ESSENTIAL PERFORMANCE is lost or degraded due to EM DISTURBANCES (the defined term "ESSENTIAL PERFORMANCE" need not be used).

1. All necessary instructions for maintaining BASIC SAFETY and ESSENTIAL PERFORMANCE with regard to electromagnetic disturbances for the expected service life.
2. Guidance and manufacturer's declaration -electromagnetic emissions and Immunity

Table 1

Guidance and manufacturer's declaration - electromagnetic emission	
Emissions test	Compliance
RF emissions CISPR 11	Group 1
RF emissions CISPR11	Class B
Harmonic emissions IEC 61000-3-2	(N/A)
Voltage fluctuations /flicker emissions IEC 61000-3-3	(N/A)

Table 2

Guidance and manufacturer's declaration - electromagnetic Immunity		
Immunity Test	IEC 60601-1-2 Test Level	Compliance Level
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air
Electrical fast transient/burst IEC 61000-4-4	Power supply lines: ±2 kV Input/Output lines: ±1 kV	N/A
Surge IEC 61000-4-5	line(s) to line(s): ±1 kV. line(s) to earth: ±2 kV. 100 kHz repetition frequency	N/A
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0% 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0% 1 cycle And 70% 25/30 cycles Single phase: at 0 0% 300 cycle	N/A
Power frequency magnetic field IEC 61000-4-8	30 A/m, 50 Hz / 60 Hz	30 A/m, 50 Hz / 60 Hz
Conducted RF IEC61000-4-6	150KHz to 80MHz: 3Vrms 6Vrms (in ISM and amateur radio bands) 80% Am at 1kHz	N/A

Radiated RF IEC61000-4-3	150KHz to 80MHz: 3Vrms 6Vrms (in ISM and amateur radio bands) 80% Am at 1kHz	N/A
Note: U T is the a.c. mains voltage prior to application of the test level.		

Table 3 - Guidance and manufacturer's declaration - electromagnetic Immunity

Radiated RF IEC61000-4-3 (Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment)	Test Frequency (MHz)	Band (MHz)	Service	Modulation	Power (W)	Distance (m)	Immunity Test Level (V/M)
	385	380 – 390	TETRA 400	Pulse modulation 18 Hz	1,8	0.3	27
	450	380 – 390	GMRS 460, FRS 460	FM $\pm$ 5 kHz deviation 1 kHz sine	2	0.3	28
	710	704 – 787	LTE Band 13, 17	Pulse modulation 217 Hz	0,2	0.3	9
	745						
	780						
	810	800– 960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation 18 Hz	2	0.3	28
	870						
	930						
	1720	1700– 1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modulation 217 Hz	2	0.3	28
	1845						
	1970						
	2450	2400– 2570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation 217 Hz	2	0.3	28
	5240	5100– 5800	WLAN 802.11 a/n	Pulse modulation 217 Hz	0,2	0.3	9
	5500						
	5785						

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