WTN570

NON-CONTACT INFRARED THERMOMETER
Dijital Projeksiyonlu Temassız Termometre

ATEŞ ÖLÇEN CİHAZLAR

USER MANUAL EN

KULLANIM KILAVUZU TR

www.weewell.com
info@weewell.com

Dear Stakeholders,

We congratulate you on having one of the Weewell products which are safely preferred all over the World and becoming a part of the Weewell family. We are pleased to serve you with our carefully designed products that appeal to babies such as video monitor, audio monitor, **thermometer**, nursing and feeding group products. Please read the user manual & **SAFETY INSTRUCTIONS** carefully before using the product.

Best Regards,
Weewell Turkey Team.

TABLE OF CONTENTS

I.Product Description	1
II.Intended Use	1
III.Product Features	1
IV.Safety Precautions	1
V.Using the Thermometer	3
Taking a Measurement	4
Temperature Mode	5
Temperature Unit	6
Viewing Data Memory	6
Deleting Data Memory	7
Setting Alarm Temperature	7
Beeper Setting	7
Overall offset value setting	8
Projection function setting	8
Battery Low Voltage Detection	9
Changing the Battery.....	9
VI.Technical Specifications	9
VII.Maintenance of the Product	10
VIII.Cleaning and Disinfecting Instructions	10
IX.Accessories Included	12
X.Guidelines	12
XI.Troubleshooting	12
XII.EMC Declaration	14
XIII.Explanation of Symbols.....	16

I.Product Description

The WTN570 Non-Contact InfraRed Thermometer is a professional grade thermometer that measures body (forehead), surface and room/ambient temperature instantly and accurately. The WTN570 Non-Contact InfraRed Thermometer can be used as a non-invasive screening tool in a multi-patient environment.

II.Intended Use

The device is an infrared thermometer intended to measure forehead temperature of infants and adults without contacting human body. It can be used by consumers in household environment and doctor in clinic as reference.

III.Product Features

- Infrared sensor accurately reads body temperature 2" to 2.4" (5 to 6 cm) from forehead
- Large, 3-color LCD
- Stores the last 32 temperature readings
- Visual/audible alarm for high temperature readings
- Temperature displays in either Fahrenheit or Celsius
- Performs up to 100,000 readings
- Under body and surface mode, the color of projection is orange on human body or object

IV.Safety Precautions

Follow the maintenance advice specified in this instruction manual.

- This device must only be used for the purposes described in this

instruction manual.

- This device must only be used in an ambient temperature range of between 10°C ~ 40°C (50°F ~ 104°F).
- Always keep device in a clean, dry area.
- Do not expose the thermometer to electric shocks.
- Do not expose the thermometer to extreme temperature conditions.
- Do not use this device in relative humidity higher than 85%.
- Do not touch the infrared sensor cup and the lens with your fingers.
- Clean the infrared sensor cup and the lens with a cotton swab lightly moistened with 95% isopropyl alcohol.
- Follow the manufacturer's recommended cleaning and disinfecting instructions.
- Do not expose the thermometer to sunlight or to water.
- Do not drop the thermometer.
- Do not attempt to repair this device yourself.
- The device is not suitable for use in the presence of flammable anesthetic mixtures with oxygen or nitrous oxide.
- The device does not require calibration.
- The device is not repairable and contains no user serviceable parts.
- Modification of the thermometer is prohibited and will invalidate the warranty.
- Remove batteries if thermometer will not be used for a long period of time.
- Check that the thermometer functions safely and is in proper working condition before each use.
- Contact the manufacturer for assistance or reporting.
- Keep the thermometer out of reach of children and pets.

The WTN570 Non-Contact InfraRed Thermometer is pre-set at the factory.

It is not necessary to calibrate the device when starting it up. However, in order to obtain reliable results, you are advised to allow the thermometer to acclimate to room/ambient temperature for 15 to 20 minutes before using each time there is a significant change in environmental temperature. It is also important to allow 3 to 5 seconds intervals between measurements.

V.Using the Thermometer

The type BF applied part: Sensor.

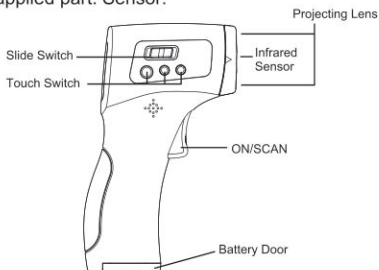
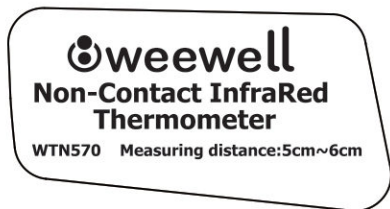
Fig1

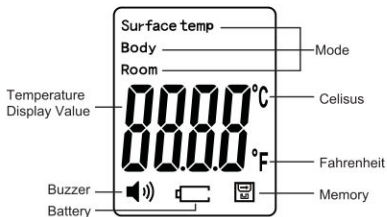
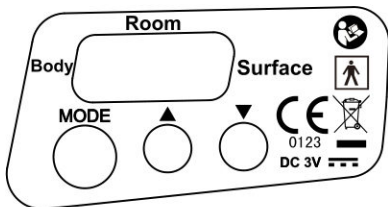
Fig.2

Taking a Measurement

1. Press “” button to power on the thermometer. When the device is ready, the LCD screen will turn green.
2. Measuring a temperature in a distance of 2" to 2.4" (5 to 6 cm) away from the patient's forehead. (Fig.3)

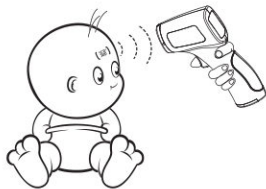
Fig.3

(36.8 °C)

Correct — Thermometer keeps a right distance and it is perp to the middle of forehead

3. Three states of projection display

Fig.4

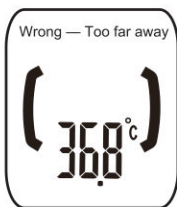
Fig.5

Fig.6

NOTE: Move hair out of the way before taking a measurement.

NOTE: If the forehead area is or has been covered with sweat, makeup or a hat, first clean the area then wait 7 to 10 minutes before taking a measurement.

3. Press “” button to take measurement.

4. A measurement will be displayed on the LCD screen.

- To repeat the process, return to step 2.
- Allow 3 to 5 seconds between measurements.

5. The thermometer automatically turns off after 30 seconds of inactivity.

Temperature Mode

The WTN570 Non-Contact InfraRed Thermometer can take body, surface or room/ambient temperature.

1. While the thermometer is turned on, Through the slide button to switch three temperature mode: Body, Room or Surface.

NOTE: When first using the thermometer, the default is set to Body mode.

NOTE: Measurement ranges for each temperature mode are as follows:

Body mode: 32°C to 43°C (89.6°F to 109.4°F)

Surface mode: 0°C to 60°C (32°F to 140°F)

Room mode: 0°C to 40°C (32°F to 104°F)

NOTE: Once the thermometer is powered on, the last temperature mode measured will default as the current temperature mode.

NOTE: Always use the Body mode to measure internal human temperature.

NOTE: When measurement is in environment temperature < 10°C or > 40°C, LCD reads ERR1 and beeps 2 sounds at the same time.

Temperature Unit

In standby mode, press “MODE” button for 2 seconds, the screen shows **F-1** function, press “▲” button is Fahrenheit, press “▼” button is degree Celsius, then press “MODE” button for confirm.

NOTE: The WTN570 Non-Contact InfraRed Projection Thermometer default setting is Celsius.

Viewing Data Memory

1. While the thermometer is turned on, press “▲” and “▼” button at the same time for 3 seconds or above, the last history data can be checked.
2. Press “▲” button again shows the last send data and so on. It can show the latest 32 sets data, it exit the mode and power-off when it without any operation for 30 seconds.

Deleting Data Memory

1. In standby status, long press “▼” button for 3 seconds above, all the memories will be cleared.
2. Press any other button to exit Mode setting or wait 30 seconds for thermometer to automatically power off.

Setting Alarm Temperature

1. While the thermometer is turned on, press the button “MODE” for 2 seconds. The screen will display **F-1**.
2. Press “MODE” button again enter to F2 function. Press the “▲” button to increased 0.1°C, press the “▼” button to decrease 0.1°C.
3. Confirm by pressing “MODE” button .

NOTE: If the temperature taken is < the setting alarm value, the buzzer sends out 1 sound and reports the current temperature in a green backlit screen. When the temperature is ≥ the setting alarm value, the buzzer sends out 5 sounds and reports the current temperature in a red backlit screen.

NOTE: For temperature unit Fahrenheit, the alarm range is 96.8°F to 109.4°F. If the temperature unit is Celsius, the alarm range is 36°C to 43°C.

NOTE: High temperature alarm default setting is 38°C(100.4°F).

Beeper Setting

1. While the thermometer is turned on, press “MODE” button for 2 seconds. The screen will display **F-1**.
2. Press “MODE” button for twice enter to F3 function, press the “▲” button to open the beeper (a trumpet symbol “”) flashes on the

screen and shows **ON**); press the “▼” button to close the beeper (a trumpet symbol “”) flashes on the screen and shows **OFF**); then press “MODE” button to confirm the setting. The sign of “” disappears on the screen.

3. To exit Beeper setting, press any other button or wait 30 seconds for thermometer to automatically power off.

Overall offset value setting

1. While the thermometer is turned on, press “MODE” button for 2 seconds. The screen will display **F-1**.
2. Press “MODE” button for three times enter to F4 function, press the “▲” button to increased 0.1°C, press the “▼” button to decrease 0.1°C.
3. Confirm by pressing “MODE” button.

NOTE: Overall offset setting's maximum adjustable range is -3.0~3.0°C, If the temperature unit is Fahrenheit, the alarm range is -5.4~5.4°F.

NOTE: Overall offset value setting is 0°C.

Projection function setting

1. While the thermometer is turned on, press “MODE” button for 2 seconds. The screen will display **F-1**.
2. Press “MODE” button for four times enter to F5 function. Press “▲” button to turn on projection function. Press “▼” button to turn off projection.
3. Confirm by pressing “MODE” button .

NOTE: Projection function default setting is projection ON.

NOTE: Under body and surface mode, the color of projection is orange on human body or object. Room mode has no projection function.


Battery Low Voltage Detection

If the battery voltage is no more than 2.7V, the WTN570 Non-Contact InfraRed Thermometer will display the “” flashed symbol to remind you to replace both batteries.

Changing the Battery

The WTN570 Non-Contact InfraRed Thermometer comes with 2 AA alkaline batteries.

When the LCD screen displays “”, both batteries should be replaced. Open the lid on the underside of the thermometer and replace the batteries, taking great care with the correct battery positioning. (Fig.7)

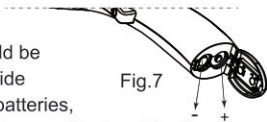VI. Technical Specifications

Device Type	Non-Contact InfraRed Thermometer
Measurement Distance	2.0 to 2.4 inches(5 to 6 cm) from forehead
Measuring Range	Body temperature:32.0°C ~ 43.0°C(89.6°F ~ 109.4°F) Surface temperature:0°C ~ 60°C(32.0°F ~ 140.0°F) Room temperature:0°C ~ 40°C(32.0°F ~ 104.0°F)
Display Type	Large, 3-color LCD
Display Resolution	0.1°C(0.1°F)
Accuracy	32.0°C ~ 34.9°C(89.6°F ~ 94.8°F); ±0.3°C(±0.6°F) 35.0°C ~ 42.0°C (95°F ~ 107.6°F); ±0.2°C(±0.4°F) 42.1°C ~ 43.0°C(107.8°F ~ 109.4°F); ±0.3°C(±0.6°F)
Operating Temperature	10°C to 40°C(50°F to 104°F)
Operating Humidity	Up to 85% non condensing
Storage Temperature	-20°C to 55°C(-4°F to 131°F)
Storage Humidity	Up to 95% non condensing
Pressure Altitude	700hPa~1060hPa
Consumption	≤300mW
Memory	Store last 32 readings
Automatic Power Off	30 seconds after last measurement
Batteries	2 AA batteries
Unit Weight	164g with batteries
Unit Size	153×93×45mm(6.02×3.66×1.77in) L×W×H
Warranty	2 years

NOTE: The WTN570 Non-Contact InfraRed Thermometer can take temperature readings below 32°C or above 43°C (89.6°F or above 109.4°F) but precision is not guaranteed outside of this range.

Longevity of the Product The WTN570 Non-Contact InfraRed Thermometer was conceived for professional use and should provide approximately 100,000 measurements.

VII.Maintenance of the Product

- The infrared sensor is the most important and fragile part of the thermometer. Take care when using, storing or cleaning.
- Clean and/or disinfect the thermometer according to your facility's infection control protocol.
- Clean infrared sensor cup and two lens with cotton swab dampened with 95% isopropyl alcohol solution.
- Please dispose of batteries properly.
- Remove the battery when thermometer is not used for an extended period of time.
- Do not expose the thermometer to water or excessive sunlight.
- Any impact to the thermometer may damage the product.

VIII.Cleaning and Disinfecting Instructions

The thermometer can be cleaned and an intermediate-level of disinfection can be achieved using the following method:

CAUTION

Never submerge the thermometer in water or any other liquid.

 **CAUTION** Never use abrasive cleaning agents, thinners or benzene for cleaning and never immerse the instrument in water or other cleaning liquids.

 **CAUTION** Never insert a sharp object into the scanner area or any other open surface on the thermometer.

 **CAUTION** Do not use unapproved cleaning or disinfection agents. Use of these agents may cause damage to components.

 **CAUTION** Do not use chemicals other than isopropyl or ethyl alcohol on the lens.

Cleaning the Sensor Window

Slightly moisten a cotton swab or cloth with isopropyl (95% minimum) alcohol and gently wipe the surface of the lens using a side-to-side (not circular) motion. Avoid touching the lens except when cleaning is required.

Cleaning the Thermometer

Use a pre-moistened wipe or soft cloth slightly moistened with an approved cleaner to gently clean the thermometer.

Disinfecting the Thermometer

Use a pre-moistened wipe or soft cloth slightly moistened with an approved disinfectant to disinfect the thermometer. Follow the manufacturer's recommendations. After cleaning or disinfecting, wait at least 8-10 minutes before taking another measurement.

IX. Accessories Included

- User manual: 1 pc
- AA alkaline batteries: 2 pcs
- Carry Bag: 1 pc

X. Guidelines

This device complies with the EU Directive 93/42/EEC concerning medical products, the ISO 80601-2-56 and the European Standard EN60601-1-2 and is subject to particular precautions with regard to electromagnetic compatibility.

XI. Troubleshooting

If you have problems while using the WTN570 Non-Contact InfraRed Thermometer, please refer to this guide to help resolve the problem.

If the problem persists, please contact our customer service.

1. The WTN570 Non-Contact InfraRed Thermometer can display temperature in either Fahrenheit or in Celsius.
2. Under Body mode:
 - When the measured value is $< 32^{\circ}\text{C}$ (89.6°F), the thermometer beeps 1 sound and displays **Lo**.
 - When the measured value is $\geq 32.0^{\circ}\text{C}$ (89.6°F) and $\leq 43.0^{\circ}\text{C}$ (109.4°F), if the value is $\geq$ setting alarm temperature, the thermometer beeps 5 sounds and displays the temperature; if the value is $<$ setting alarm temperature, the thermometer beeps 1 sound and displays the temperature.
 - When the measured value is $> 43.0^{\circ}\text{C}$ (109.4°F), the thermometer beeps 5 sounds and displays **Hi**.

3. Under the Surface temp mode:

- When measured value is $< 0^{\circ}\text{C}$ (32°F), the thermometer beeps 1 sound and displays **Lo**.
- When the measured value is $\geq 0^{\circ}\text{C}$ (32°F) and $\leq 60^{\circ}\text{C}$ (140°F), the thermometer beeps 1 sound and displays the temperature.
- When the measured value is $> 60^{\circ}\text{C}$ (140°F), the thermometer beeps 1 sound and displays **Hi**.

4. Under the Room mode:

- When measured value $< 0^{\circ}\text{C}$ (32°F), the thermometer beeps once and displays **Lo**.
- When the measured value $\geq 0^{\circ}\text{C}$ (32°F) and $\leq 40^{\circ}\text{C}$ (104°F), the thermometer beeps 1 sound and displays the temperature.
- When the measured value $> 40^{\circ}\text{C}$ (104°F), the thermometer beeps 1 sound and displays **Hi**.

Reasons for Lo message display	Advice
Temperature reading is hampered by hair or perspiration.	Make sure there is no obstruction or dampness prior to taking temperature.
Temperature hampered by an air draft or dramatic change in ambient temperature.	Make sure there is no air blowing in the area of use, this could affect the infrared reading.
Temperature readings are too close together, and the thermometer did not have time to reboot.	Pause for 3~5 seconds minimum between two readings; a 15 minutes pause is recommended.
Measuring too far from the forehead.	Take measurements at the recommended distance of 2" to 2.4" (5 to 6 cm) from forehead.
The ambient temp $< 0^{\circ}\text{C}$ (32°F)	Make sure measuring environment temperature is higher than 0°C (32°F).
Measuring after cold compress/taking medicine/ bathing.	If any of these apply, make sure to wait 30 minutes and test again.

Reasons for Hi message display	Advice
When body temperature is above 43°C (109.4°F).	Temperature reading exceeds measurement range of 32°C to 43°C (89.6°F to 109.4°F). Make sure there is no hot air blowing in the area of use. If necessary, immediately seek medical attention.
The F2 value is on the high side.	Reset high alarm temperature setting closer to or at default value of 38°C (100.4°F).
The room/ambient temperature is above 40°C (104°F).	Make sure to operate thermometer between 10°C to 40°C (50°F to 104°F).
The surface temperature exceeds 60°C (140°F).	Temperature reading exceeds measurement range of 0°C to 60°C (32°F to 140°F).

XII. EMC Declaration

Guidance and manufacturer's declaration – electromagnetic immunity			
The "WTN570" is intended for use in the electromagnetic environment specified below. The customer or the user of the "WTN570" should ensure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment –guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines	Not Applicable	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV line(s) to line(s) ± 2 kV line(s) to line(s)	Not Applicable	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5 % U _n (>95 % dip in U _n) for 0.5 cycle 40 % U _n (60 % dip in U _n) for 5 cycles 70 % U _n (30 % dip in U _n) for 25 cycles <5 % U _n (>95 % dip in U _n) for 5 sec	Not Applicable	Mains power quality should be that of a typical commercial or hospital environment. If the user of the "WTN570" requires continued operation during power mains interruptions, it is recommended that the "WTN570" be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE: U _n is the a.c. mains voltage prior to application of the test level.			

Guidance and manufacturer's declaration – electromagnetic immunity

The "WTN570" is intended for use in the electromagnetic environment specified below. The customer or the user of the "WTN570" should ensure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	Not Applicable	Portable and mobile RF communications equipment should be used no closer to any part of the "WTN570", including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = 1.167 \sqrt{P}$ $d = 1.167 \sqrt{P}$ 80MHz to 800MHz $d = 2.333 \sqrt{P}$ 800MHz to 2.5 GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, ^a should be less than the compliance level in each frequency range. ^b Interference may occur in the vicinity of equipment marked with the following symbol: 
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	

NOTE 1: At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

^a

Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the "WTN570" is used exceeds the applicable RF compliance level above, the Medical WTN570 should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the "WTN570".

^b

Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Recommended separation distances between portable and mobile RF communications equipment and the Medical WTN570

The "WTN570" is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Medical WTN570 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the "WTN570" as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1.167 \sqrt{P}$	80 MHz to 800 MHz $d = 1.167 \sqrt{P}$	800 MHz to 2.5 GHz $d = 2.333 \sqrt{P}$
0.01	0.117	0.117	0.233
0.1	0.369	0.369	0.738
1	1.167	1.167	2.333
10	3.690	3.690	7.377
100	11.67	11.67	23.33

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

XIII.Explanation of Symbols

	3V DC Power Supply
	IEC 60417-5333, Type BF applied part
	In order to protect the environment, please recycle the battery according to the local regulations
Body Room Surface	Indication of Mode
	<u>Attention consult accompanying documents</u>
	Refer to instruction manual / booklet
	This symbol shall be accompanied by the name and the address of the manufacturer.

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Guangzhou Berrcom Medical Device co.,Ltd.

**Add.: No.38 Huanzhen Xi Road, Dagang Town, Nansha, 511470 Guangzhou,Guangdong,
PEOPLE'S REPUBLIC OF CHINA
Tel:+8620-3480 1318 Fax:+8620-3480 2683**

Declaration of Conformity

We , **Guangzhou Berrcom Medical Device co.,Ltd.** Add.:**No.38 Huanzhen Xi Road, Dagang Town, Nansha, 511470 Guangzhou,Guangdong, PEOPLE'S REPUBLIC OF CHINA.**
declare under our sole responsibility that the product (s) :

Description : Non-Contact Infrared Thermometer

Brand name : WEEWELL

Model Number : WTN550R&WTN570

Classification: Class IIa

EC Representative:Wellkang Ltd

Add.: The Black Church, St. Mary's Place, Dublin 7, D07 P4AX, Ireland

To which this declaration relates is in conformity with the essential requirements and other relevant requirements of MDD Directive 93/42/EEC. The product is in conformity with the following standards and/or other normative documents :

IEC 60601-1:2005+A1:2012 / EN60601-1:2006+A1:2013

IEC 60601-1-1:2014 / EN 60601-1-2:2015

IEC 62471: 2006 / EN 62471: 2008

IEC 60601-1-11:2015 / EN 60601-1-11:2015

Manufacturer : Guangzhou Berrcom Medical Device co.,Ltd.

Add.: No.38 Huanzhen Xi Road, Dagang Town, Nansha, 511470 Guangzhou,Guangdong,
PEOPLE'S REPUBLIC OF CHINA

Tel:+8620-3480 1318 **Fax:**+8620-3480 2683

Notified Body: TUV SUD Product Service GmbH

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Model : WTN550R&WTN570

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AT Temsilcisi : Wellkang Ltd

Adres : The Black Church, St. Mary's Place, Dublin 7, D07 P4AX, Ireland

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IEC 60601-1-11:2015 / EN 60601-1-11:2015

Üretici : Guangzhou Berrcom Medical Device co.,Ltd.

Adres : No.38 Huanzhen Xi Road, Dagang Town, Nansha, 511470 Guangzhou,Guangdong,
PEOPLE'S REPUBLIC OF CHINA

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